



Conus Denture Flow Chart

First Appointment (4 Days):

- * Send to lab study cast or impression.
- * Request: custom tray and impression copings (please provide info on implants).

Second Appointment (4 Days):

- * Take a fixture-level impression using custom tray.
- * Request: base plate with wax rim and verification jig.

Third Appointment (3 Days):

- * Take registration, shade and try-in the verification jig and ensure that they align. Do NOT lute together. If they do not align, we will need a new fixture-level impression as the model work will be incorrect.
- * Request: set up in wax.

Fourth Appointment:

- * Try in set up, confirm set up.
- * Request process (**10 Days Process**) and Conus Gold cylinders or Reset in wax (**3 Days Reset**)

LAB PROCESSES

Fifth Appointment:

- * Place final Conus Abutments using placement jig provided by laboratory.
- * Insert denture, adjust occlusion.

OR

DOCTOR PROCESSES (cylinders in mouth)

Fifth Appointment:

- * Place final Conus Abutments using placement jig provided by laboratory.
- * Trial fit denture over abutments Gold cylinders.
- * Lute the cylinders in place with auto-polymerizing resin or dual cure cement.
- * Clean flash around cylinders.
- * Adjust occlusion.

For more information, contact Steve Daggett or Dennis McPherson at 800.236.3859.