



Conus Denture Flow Chart

*** Pick up in the morning counts as first day in lab; pick up in the afternoon counts as next day in lab*

First Appointment

- Send to lab study cast or impression
- Request custom tray and impression copings, providing info on implants (4 days in lab)

Second Appointment

- Take a fixture-level impression using custom tray
- Request base plate with wax rim and verification jig (4 days in lab)

Third Appointment

- Take registration, shade, and try-in the verification jig to ensure that they align – do NOT lute together (if they do not align, we will need a new fixture-level impression as the model work will be incorrect because the abutments will be engaging)
- Request set-up in wax (7 days in lab)

Fourth Appointment

- Try-in set-up and confirm
- Request processing (10 days in lab) and Conus Gold cylinders or reset in wax (3 days in lab)

Fifth Appointment

- Place final Conus abutments using placement jig (provided by lab)
- Trial fit denture over abutments Gold cylinders
- Lute the cylinders in place with auto-polymerizing resin or dual cure cement
- Clean flash around cylinders
- Adjust occlusion

Please contact us with any questions at (800) 236-3859