


# ATTENTION! DOCTORS

## Care and cleaning for this Lucitone digital denture is different than traditional dentures.

- Patients SHOULD NOT store digital dentures in water, mouthwash or an overnight cleaning solution. Store dry in a container.
- Patients can brush their digital denture with a soft bristle toothbrush using a mild soap like Ivory. They can also use popular denture cleansers like Polident or Efferdent – just don't soak the denture overnight in the cleaning solution or in water.
- Like traditional dentures, avoid hot or boiling water as this may warp the digital denture.

Please provide your patient with the enclosed Care and Cleaning Instructions.

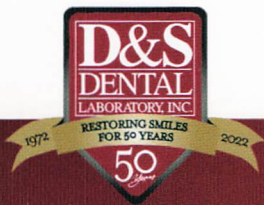

[DNSDENTAL.COM](http://DNSDENTAL.COM) | 800.236.3859